

Qivi Male External Urine Management

Bedside Tip Sheet

- Clean and dry the anatomy prior to device application
- Trim the pubic hair to get a superior seal around the anatomy
- For patients with high BMI, push the pannus down to get access to the full shaft
- Set the wall suction to at least 40mmHg and adjust if needed
- Ensure the suction tubing is connected with Qivi at all times
- Replace the device every 12-24 hours



PRODUCT USE VIDEO

1800 520 4714

sales@consuremedical.com

www.consuremedical.com

