


PREPARATION

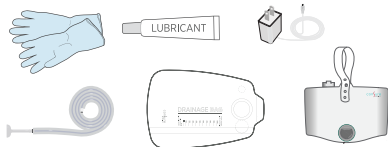

In addition to Qoramatic™ Stool Management Kit, lubricant, gloves, and 500ml of distilled water or saline will be required.

Qoramatic



Stool Management Kit

DRAINAGE BAG SETUP




Fill Irrigation Chamber

Use 500ml of distilled water or saline to fill drainage bag irrigation chamber.

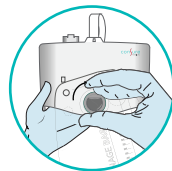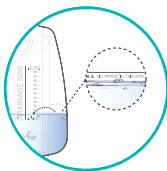


Connect Bag

Mate device and bag connector. Twist clockwise to secure connection.

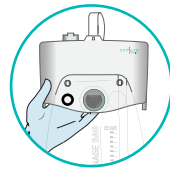

Connect Irrigation


Mate drainage bag irrigation connector to the matic hub irrigation port.

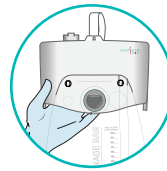

Secure Bag


Tether eyelets onto the hooks of matic hub.

MATIC HUB SETUP

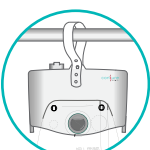

Hang Device

Loop device strap over the bedside rail and secure. Ensure device is upright.

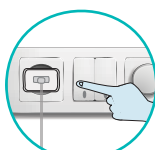

Connect Power Adapter


Connect power adapter with the Matic Hub. A solid red indicator will appear on the device.

RECEPTACLE INSERTION




Fold and lubricate

Fold receptacle to create a cone and generously apply lubricant.

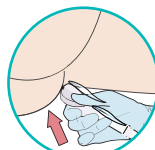

Insert


Gently insert receptacle till it is past anal canal and above anorectal junction.

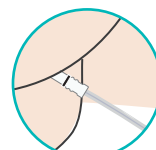

Ensure Correct Placement

Black receptacle indicator visible immediately outside anal orifice indicates correct device placement.

RECEPTACLE CONNECTION

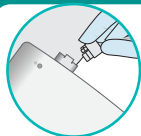


Connect Receptacle

Mate connector at receptacle distal end with matic hub. Ensure snug fit. Push start button to turn on the device. Red indicator will turn green.

MAINTENANCE

Replace Bags


Drainage bag to be replaced when full or when void of irrigation water, typically every 8-10 hours. To unmate bag, unplug power adapter. Tilt matic hub backwards by 45 degrees to prevent any spillage. Remove bag from hooks, disconnect irrigation connector and twist bag connector in anti clockwise direction to unmate bag. Use the instructions under drainage bag setup to replace bag.

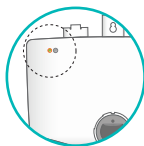

Check Indicator

Check device indicator periodically to ensure consistent device operation.

REMOVAL AND DISPOSAL

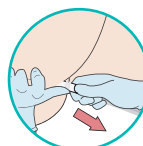

Remove Receptacle

Unplug the device. Hold the receptacle close to anal orifice and retrieve slowly.



Device Disposal

Discard device according to institute protocol.

QORAMATIC STATUS INDICATORS

Colour	Device status	Action required
● Solid Red	Power on	Press start button to initiate operation
● Solid Green	Device in rest	No user input required.
⚡ Blinking Green	Device in operation	No user input required. (Do not replace bag when device in operation.)
● Solid Blue	Water error	Ensure irrigation connector on drainage bag and irrigation port on matic hub are securely connected. Top-up irrigation fluid or replace the drainage bag.
● Solid Magenta	Tube block error	Press start button. If error persists, replace the device with a new unit.
⚡ Blinking Red	Suction error	Try one of the following, and then press start button: 1. Ensure receptacle is properly placed inside patient rectum. 2. Ensure drainage bag and receptacle unit connections are properly secured to the matic hub. 3. Replace drainage bag with a new bag.